

## Instructions:



Clean Fusion bur box in running water.



Pour the disinfectant Dento-Viractis 57 directly into the Fusion bur box.



Immerse the Bur Steri Com and let soak for 15 minutes.



Clean it in Ultrasonic cleaner (Elmasonic S10).



Dry with a tissue paper.



Seal the Bur Steri Com with 100mm sterilization pouch roll.



Autoclave the Fusion bur box in the autoclave (Ficoinox H12L).



Ready to use.



**Bur Steri Com**

Article No. 690032

### Features of Fusion Bur Organiser Set

- Fusion is an ideal bur dispensing and autoclaving box.
- It can achieve efficient cleaning and sterilizing dental burs.
- Available in 4 colours for different procedures (preparation, cleaning & polishing)



**Fusion Bur Organiser Set**

Article No. 690033

### Contents of Fusion Bur Organiser Set

- Bur Steri Com (FG) Blue - 2 Pcs
- Bur Steri Com (FG) Orange - 1 Pc
- Bur Steri Com (FG) Red - 1 Pc
- Bur Steri Com (CAHP) Grey - 1 Pc
- Outer Box (Transparent) - 1 Pc

Manufactured By:

**Dispodent**

No. 2, Bharathi Street, Ponniamma Medu, Chennai - 600 110. INDIA.

Customer Care: 044 - 2556 0669 info@dispodent.com / www.dispodent.com